

# PRODUCT ALTERNATIVE

# MEDICHOICE® Ultrasound Gel Premium High Viscosity

- Bacteriostatic Water Soluble Perfume Free Paraben Free
- Formulated to Ensure Minimal Air Bubbles
- For Diagnostic and Therapeutic Ultrasound Procedures

### 1 Fach Reorder: 10080212

#### **Instructions for Use:**

- 1. Flip open cap.
- 2. Squeeze out and apply desired amount of gel before scanning. Tip should not come in contact with the patient's skin.
- 3. Close cap after use.
- 4. After procedure, remove gel with a slightly damp cloth or paper towel.



(01)10885632133367 (17)200203 (10)160133

2020-02-03 LOT 160133

Distributed by Owens & Minor 9120 Lockwood Boulevard Mechanicsville, VA 23116

Net Wt. 8 oz. (226 g)

70076D



If irritation or discomfort occurs, discontinue use and consult a physician

Not for use with automated external defibrillators.

Keep out of reach of children

Federal law restricts this device to sale by or on the order of a physician. Non-sterile. For external use only Contains carbomers.



## MEDI**CHOICE**®

# Ultrasound Gel

- Bacteriostatic Salt Free
- Water Soluble
- Non-Greasy
- Thick No-Slip Formula
  - · Reduces Likelihood Of Skin Irritation

### 1 Each

### Reorder: M500812





#### Instructions for Use:

- 1, Flip open cap. Remove protective liner,
- 2. Squeeze out and apply desired amount of gel before scanning. Tip should not come into contact with patient's skin,
- 3. Close cap after use.
- 4. After procedure, remove gel with a slightly damp cloth or paper towel. WARNING:
- . Not for use with electromedical devices.
- . If irritation or discomfort occurs, discontinue use and consult a physician. · Keep out of reach of children.

#### CAUTION:

- Federal law restricts this device to sale by or on the order of a physician.
- . Non-sterile. For external use only.
- Contains carbomers.

Distributed by Owens & Minor 9120 Lockwood Boulevard Mechanicsville, VA 23116 Made in Canada

Net Contents 8.5 fl.oz.

Rev # 2016-1

